

MED DIMENSIONS

COCKATOO INTUBATION

MD-E-A1021S



User Manual

Intended Use


1. Use our intubation trainer to practice inserting an intubation tube into the syringe.
 - Near unlimited use

1. Parts Included

1. Tongue
2. Airway
3. Hard case body
4. Flexible jaw

2. Tools Recommended

1. Size 3.5 endotracheal tube
2. Lubricant

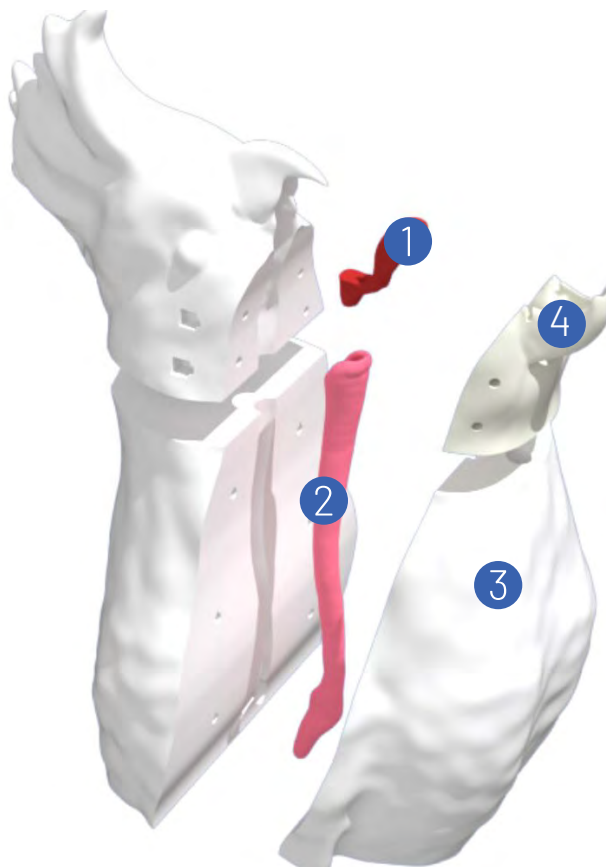

3. Assembly Instructions

1. **To View the Syrinx:** Hold the back of the head and body firmly. With your other hand, grab the abdomen and twist counterclockwise or clockwise to disengage the magnets.
2. Lubricate as you would in a normal setting. Flush with water to remove lubricant as needed.



Best Practices

To keep your models in best working order we recommend the following:

1. Store your model in a cool, dark, and low dust location.
2. For models containing or exposed to water, wipe down excess, leave all connectors open and allow components to dry completely before storing.
3. If applicable, apply Silicone Finishing Powder to reduce tack and dust accumulation.
4. If applicable, flush or wash models with water to remove lubricant and debris.
5. Work on a flat surface to prevent the model from slipping.

Warranty Information

This is a summary of the Med Dimensions warranty policy provided for your convenience. For the complete and most up-to-date warranty information, please refer to the full policy available at <https://www.med-dimensions.com/warranty/>.

Med Dimensions warrants its products to be free from defects in design, operation, workmanship, and materials. Warranty duration varies by type of product as described on our warranty page. This warranty does not cover damage resulting from misuse, abuse, neglect, or unauthorized repair. To initiate a warranty claim, contact Med Dimensions Customer Support at support@med-dimensions.com or 585-510-0005 x1. Proof of purchase is required for warranty service. This warranty supersedes all other warranties, whether expressed or implied, including warranties of merchantability or fitness for a particular purpose. Med Dimensions is not liable for consequential or incidental damages arising from product use or inability to use.