

MED DIMENSIONS

RABBIT INTUBATION

MD-E-R1021S



User Manual

Intended Use


1. Use our intubation trainer to practice inserting an intubation tube or laryngoscope.
 - Near unlimited use

1. Parts Included

1. Head and teeth
2. Jaw and teeth
3. Soft palate
4. Tongue with epiglottis and pharynx

2. Tools Recommended

1. Size 2.5 endotracheal tube
2. Lubricant

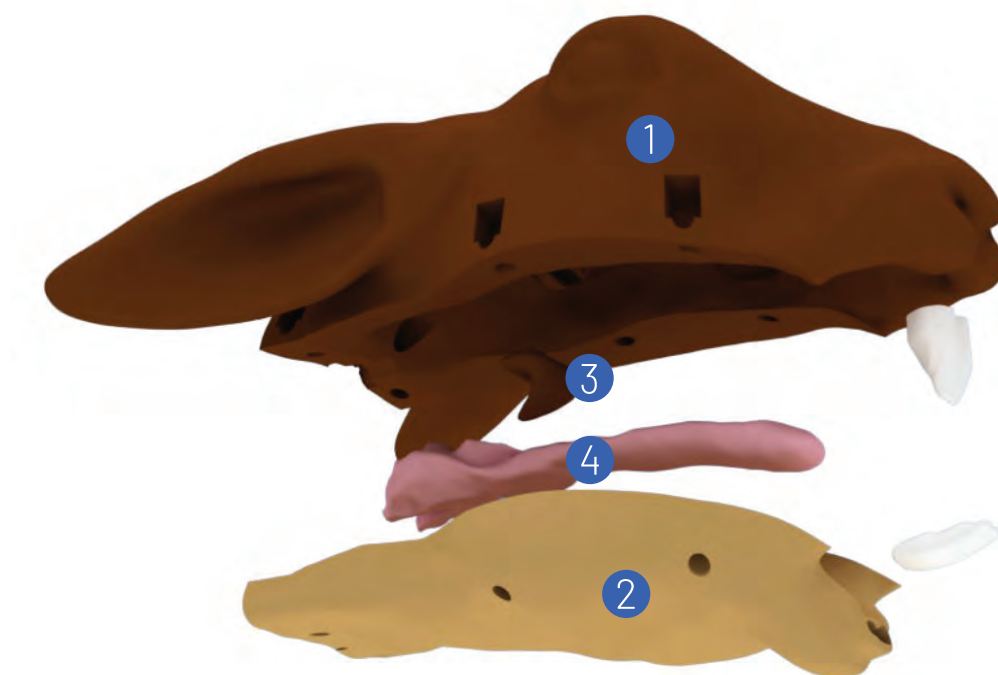

3. Assembly Instructions

1. No assembly required. If the teeth become displaced during shipping, simply press them back into the jaw or head.
2. Lubricate as you would in a normal setting. Flush with water to remove lubricant as needed.



Best Practices

To keep your models in best working order we recommend the following:

- 1. Store your model in a cool, dark, and low dust location.
- 2. For models containing or exposed to water, wipe down excess, leave all connectors open and allow components to dry completely before storing.
- 3. If applicable, apply Silicone Finishing Powder to reduce tack and dust accumulation.
- 4. If applicable, flush or wash models with water to remove lubricant and debris.
- 5. Work on a flat surface to prevent the model from slipping.

Replacement Component

Component	SKU	Material	Size	Weight
Replacement Jaw/Tongue	MD-E-R1021S-1	Rubber, Silicone	4.5x2.5x1 in	0.1 lbs

Warranty Information

This is a summary of the Med Dimensions warranty policy provided for your convenience. For the complete and most up-to-date warranty information, please refer to the full policy available at <https://www.med-dimensions.com/warranty/>.

Med Dimensions warrants its products to be free from defects in design, operation, workmanship, and materials. Warranty duration varies by type of product as described on our warranty page. This warranty does not cover damage resulting from misuse, abuse, neglect, or unauthorized repair. To initiate a warranty claim, contact Med Dimensions Customer Support at support@med-dimensions.com or [585-510-0005](tel:585-510-0005) x1. Proof of purchase is required for warranty service. This warranty supersedes all other warranties, whether expressed or implied, including warranties of merchantability or fitness for a particular purpose. Med Dimensions is not liable for consequential or incidental damages arising from product use or inability to use.